

SalivaConfirm™ Premium

Saliva Drug Testing Kit



salivaconfirm®
oral fluid multi-drug screening kit



Transparency is key when it comes to drug testing, that's why SalivaConfirm™ Premium is unmatched. This fully integrated, non-invasive saliva drug test screens for up to 14 common drugs of abuse, and features an indicator that confirm adequate saliva collection. SalivaConfirm™ Premium eliminates the risk of adulteration inherent in other drug testing methods, since the test can be supervised by the collector without imposing privacy measures. For ease of reporting, the device also includes a flat-panel that can be photocopied once results are obtained.

TESTS FOR:

Alcohol (ALC), Amphetamine (AMP), Barbiturates (BAR), Benzodiazepines (BZO), Buprenorphine (BUP), Cocaine (COC), Fentanyl (FEN), K2 Spice, Marijuana (THC), Methadone (MTD), Methamphetamine (mAMP), Morphine/Opiates (MOR/OPI), Oxycodone (OXY), Phencyclidine (PCP)

PRODUCT FEATURES:

- Tests up to 14 drugs of abuse, including Alcohol, Fentanyl, and K2/Spice
- 5, 6, 10, 12 & 14 panel configurations
- Saliva indicator for sufficient sample collection
- Detects drug use hours before urinalysis
- Lowest drug concentration cut-offs
- Flat-panel for easy photocopying
- Non-invasive with low adulteration risk
- Results available in 10 minutes or less
- 25 tests per box

SALIVA CONFIRM™ PREMIUM ACCELERATES DRUG TESTING LIKE NO OTHER

For rapid reporting without compromising accuracy, SalivaConfirm™ Premium is a state-of-the-art drug testing solution. With lower drug concentration cut-offs than any other specimen type, saliva drug tests are increasingly popular. Our non-invasive SalivaConfirm™ Premium quickly identifies substance abuse while reducing the risk of sample tampering.

INSTRUCTIONS

1. Remove the collection stick and test tube from the sealed pouch, then label the device with donor information.
2. Insert the sponge end of the collection stick into mouth and soak sponge into saliva for 3 minutes. Note: If the amount of saliva pressed into the test tube is not adequate to initiate testing, try again with a new collection stick and use same tube.
3. Hold the test tube vertically and place the collection stick into the tube. Press the stick all the way down until the lid seals the tube. Keep the test tube upright until you begin to read results.
4. Read results of alcohol test at 3 minutes and drug tests at 10 minutes. Peel off the label to read test results.

INTERPRETATION OF RESULTS

NEGATIVE:

Two color lines appear. One color line in the Control (C) region and another line next to the test (T) region.



POSITIVE:

One color line appears in the control (C) region and no line appears in the test (T) region.



INVALID:

Control line fails to appear. In case of invalid results, review procedure and repeat test using a new test device. If problem persists, discontinue using the lot immediately and contact your supplier.



IMPORTANT INFORMATION

Do not use after the expiration date. The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or the sample is adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests confirm the preliminary results.



TEST	DESCRIPTION	CUT-OFF LEVEL
ALC	ALCOHOL	>0.02% B.A.C.
AMP	AMPHETAMINE	50 NG/ML
BAR	BARBITURATES	300 NG/ML
BZO	BENZODIAZEPINES	50 NG/ML
BUP	BUPRENORPHINE	10 NG/ML
COC	COCAINE	20 NG/ML
FEN	FENTANYL	10 NG/ML
K2/SPICE	K2 SPICE	5 NG/ML
THC	MARIJUANA	11-NOR- $\Delta(9)$ -THC-9 COOH - 12NG/ML $\Delta(9)$ -THC - 75 NG/ML
mAMP	METHAMPHETAMINE	50 NG/ML
MOR/OPI	MORPHINE/OPIATES	40 NG/ML
MTD	METHADONE	75 NG/ML
OXY	OXYCODONE	50 NG/ML
PCP	PHENCYCLIDINE	10 NG/ML