

# iCup<sup>®</sup>

## Urine Drug Test



Flat Panel for  
Photocopying



Instant Results



Difficult to  
Adulterate



iCup's CLIA-Waived, fully integrated urine drug test cup maintains 99% reporting accuracy, whether used at home or in the workplace. iCup's panels can be custom configured to detect 3 to 13 common drugs of abuse. Built-in adulteration strips monitor temperature, safeguard against specimen tampering and yield results within 5 minutes. iCup's design showcases a flat side to speed up photocopying and scanning. For one of the fastest, most advanced flat-paneled drug testing device on the market, iCup is the answer.

### TESTS FOR:

Amphetamine (AMP), Barbiturates (BAR), Benzodiazepines (BZO), Buprenorphine (BUP), Cocaine (COC), Ecstasy/Molly (MDMA), Marijuana (THC), Methadone (MTD), Methamphetamine (mAMP), Morphine/Opiates (OPI), Oxycodone (OXY), Phencyclidine (PCP), Propoxyphene (PPX), Tricyclic Antidepressants (TCA)

### PRODUCT FEATURES:

- Flat Panel for Ease of Reporting
- Built-In Adulteration With Temperature Reading
- Detects Up to 13 Common Drugs of Abuse
- CLIA-Waived and FDA 510(k) Approved
- 3-13 Customizable Panel Configuration
- 99% Report Accuracy, Results in 5 Minutes
- 25 Cups Per Box



## WHY ICUP?

iCup's All-In-One design offers seamless, efficient drug testing without sacrificing lab-grade accuracy. Its unique flat panel and built-in adulteration strips provide a sensible solution for professional reporting and documenting requirements. For a streamlined, cost-effective urine collection device, choose iCup.

## INSTRUCTIONS

1. Tear the foil bag open, remove test cup. Label the device with donor information.
2. Open test cup lid. Urinate directly into the test cup. Be sure to fill the test cup with urine sample between a minimum of 30 ml to a maximum of 110 ml. The minimum and maximum levels are marked on the cup.
3. After urine specimen has been collected, close the lid securely and return cup to specimen collector.
4. For specimen collector: Peel off label to reveal test result. Read results at 5 minutes. Results should not be read after 10 minutes.

## INTERPRETATION OF RESULTS

### NEGATIVE:

Two color lines appear. One color line in the Control (C) region and another line next to the test (T) region.



### POSITIVE:

One color line appears in the control (C) region and no line appears in the test (T) region.



### INVALID:

Control line fails to appear. In case of invalid results, review procedure and repeat test using a new test device. If problem persists, discontinue using the lot immediately and contact your supplier.



## IMPORTANT INFORMATION

Do not use after the expiration date.

The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or the sample is adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests confirm the preliminary results.

TEST	DESCRIPTION	CUT-OFF LEVEL
AMP	AMPHETAMINE	1000 nG/mL
BAR	BARBITURATES	300 nG/mL
BZO	BENZODIAZEPINES	300 nG/mL
BUP	BUPRENORPHINE	10 nG/mL
COC	COCAINE	300 nG/mL
THC	MARIJUANA	50 nG/mL
MTD	METHADONE	300 nG/mL
mAMP	METHAMPHETAMINE	1000 nG/mL
MDMA	ECSTASY/MOLLY	500 nG/mL
MOR/OPI	MORPHINE/OPIATES	300/2000 nG/mL
OXY	OXYCODINE	100 nG/mL
PCP	PHENCYCLIDINE	25 nG/mL
PPX	PROPOXYPHENE	300 nG/mL
TCA	TRICYCLIC ANTIDEPRESSANTS	1000 nG/mL