

DrugConfirm™ Advanced

14 Panel Urine Drug Test Dip Card



Introducing our new 14-Panel DrugConfirm™ Advanced Urine Drug Test Dip Card, a complete clinical drug screening solution that delivers accurate CLIA Waived results in minutes. This 14-panel dip card is first-to-market, and provides an affordable solution for healthcare professionals looking to implement comprehensive urinalysis drug testing in their practice.

TESTS FOR:


Amphetamine (AMP), Barbiturates (BAR), Benzodiazepines (BZO), Buprenorphine (BUP), Cocaine (COC), Ecstasy/Molly (MDMA), Marijuana (THC), Methadone (MTD), Methadone Metabolite (EDDP), Methamphetamine (mAMP), Morphine/Opiates (MOR/OPI), Oxycodone (OXY), Phencyclidine (PCP), Propoxyphene (PPX), Tricyclic Antidepressants (TCA)

EACH KIT INCLUDES:

- Individually Wrapped Single-Panel Urine Dip Card
- Package Insert and Instructions

PRODUCT FEATURES:

- CLIA Waived, FDA 510(k) Approved, CE Marked Available
- Detects 14 Common Drugs of Abuse
- Results in 5 Minutes or Less
- 99% Result Accuracy
- 25 Dip Cards Per Case

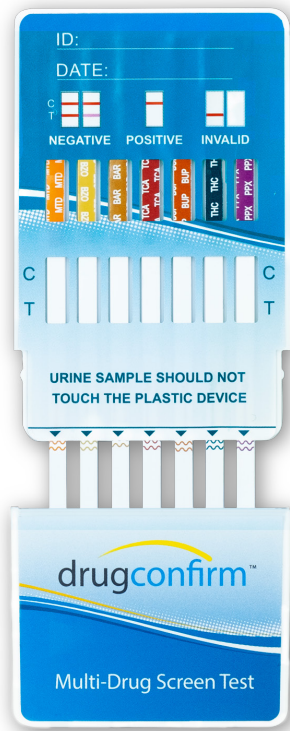

INSTRUCTIONS

1. Collect a urine sample with a clean, dry urine collection cup (not included). Bring refrigerated samples to room temperature (15°-30°C) before using test.
2. Remove test device from pouch. Label with Patient ID and use immediately.
3. Remove cap from the end of test card. With arrows pointing downward, immerse the strip portion of device into specimen for a minimum of 10-15 seconds. NOTE: For best results, DO NOT allow plastic panel to contact urine specimen.
4. Replace cap and place test card on a flat surface. Wait for colored line(s) to appear. Read result(s) after 5 minutes. DO NOT interpret test after 8 minutes or longer.

INTERPRETATION OF RESULTS

NEGATIVE:

A colored line next to the Control Region (C) and a colored line next to the Test Region (T) indicates a Negative result, regardless of color intensity.



PRELIMINARY POSITIVE:

A colored line next to the Control Region (C) and no line next to the abbreviated Test Region (T) indicates a Preliminary Positive result. Confirm Positive test results with Liquid Chromatography/Mass Spectrometry (LC/MS) method.



INVALID:

If no line appears next to the Control Region (C) after ten minutes, the result is invalid or inconclusive. Discard the drug test and administer a new one.



IMPORTANT INFORMATION

Do not use after the expiration date.

The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests are needed to confirm preliminary positives.

TEST	DESCRIPTION	CUT-OFF LEVEL
COC	COCAINE	300 NG/ML
mAMP	METHAMPHETAMINE	1000 NG/ML
MDMA	ECSTASY	500 NG/ML
MTD	METHADONE	300 NG/ML
MOR/OPI	MORPHINE/OPIATES	300/2000 NG/ML
OXY	OXYCODONE	100 NG/ML
PCP	PHENCYCLIDINE	25 NG/ML
PPX	PROPOXYPHENE	300 NG/ML
TCA	TRICYCLIC ANTIDEPRESSANTS	1000 NG/ML
THC	MARIJUANA	50 NG/ML

FOR THE MOST INCLUSIVE URINE DIP CARD IMMUNOASSAY ON THE MARKET, LOOK NO FURTHER THAN OUR DRUGCONFIRM™ ADVANCED 14-PANEL URINE DRUG TEST DIP CARDS.

With our new 14-panel urine drug test dip card, you'll be able to screen more drugs of abuse than ever before and still obtain instant, CLIA Waived results. This preliminary urine immunoassay makes substance abuse testing easier than ever before.