

DrugConfirm Advanced 3.0

+ With 80-hr EtG Alcohol
+ K2/Spice
+ Fentanyl
+ Adulterants


drugconfirm™ advanced
instant multi-drug test kit



You needed an all-inclusive urine drug test cup, and we answered the call. Introducing our DrugConfirm Advanced 3.0 Cup, which can detect the use of alcohol, synthetic marijuana, opiates, and fentanyl in addition to other common drugs of abuse. DrugConfirm Advanced 3.0 urine drug test kits feature built-in adulteration strips to ensure you receive lab-grade results within minutes. Take out the guesswork, test comprehensively, and feel confident in your chosen drug test.

TESTS FOR:

Alcohol (EtG)*, Amphetamine (AMP), Benzodiazepines (BZO), Buprenorphine (BUP), Cocaine (COC), Ecstasy/Molly (MDMA), Fentanyl (FEN)*, Marijuana (THC), Methadone (MTD), Methamphetamine (mAMP), Morphine/Opiates (MOR/OPI), Oxycodone (OXY), Synthetic Marijuana (K2/Spice)*

**For forensic use only*

PRODUCT FEATURES:

- Can detect up to 13 common drugs of abuse, K2/Spice, Fentanyl and Alcohol, plus Adulterants
- Results within minutes
- Most sensitive alcohol test on the market with 80-hour detection window
- Detects multiple K2 metabolites
- 25 cups per box

CHOOSE DRUGCONFIRM ADVANCED 2.0 FOR YOUR TESTING NEEDS



The DrugConfirm Advanced Drug Test Cup is a fully integrated, self-contained drug test kit for detecting drugs and their metabolites in urine. With DrugConfirm Advanced, you can also test for adulteration, fentanyl, K2/Spice and alcohol. The cup detects alcohol up to 80 hours after consumption, the longest detection window on the market. Our multi-panel drug screen makes donor tampering virtually impossible if an adulteration test is integrated. DrugConfirm Advanced provides an easy solution for on-site testing that avoids the handling of urine.

INSTRUCTIONS

1. Tear the foil bag open, remove test cup. Label the device with donor information.
2. Open test cup lid. Urinate directly into the test cup. Be sure to fill the test cup with urine sample between a minimum of 30 ml to a maximum of 110 ml. The minimum and maximum levels are marked on the cup.
3. After urine specimen has been collected, close the lid securely and return cup to specimen collector.
4. For specimen collector: Peel off label to reveal test result. Read test result at 5 minutes. Results expire after 10 minutes.

INTERPRETATION OF RESULTS

NEGATIVE:

Two color lines appear. One color line in the Control (C) region and another line next to the test (T) region.




POSITIVE:

One color line appears in the control (C) region and no line appears in the test (T) region.



INVALID:

Control line fails to appear. In case of invalid results, review procedure and repeat test using a new test device. If problem persists, discontinue using the lot immediately and contact your supplier.

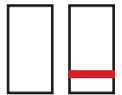

IMPORTANT INFORMATION

Do not use after the expiration date.

The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or the sample is adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests confirm the preliminary results.

TEST	DESCRIPTION	CUT-OFF LEVEL
AMP	AMPHETAMINE	1000 NG/ML
BZO	BENZODIAZEPINE	300 NG/ML
BUP	BUPRENORPHINE	10 NG/ML
COC	COCAINE	300 NG/ML
MDMA	ECSTASY	500 NG/ML
FEN	FENTANYL	200 NG/ML
EtG	ETHYL GLUCURONIDE	300/500 NG/ML
K2	K2 SYNTHETIC CANNABINOIDS	50 NG/ML
THC	MARIJUANA	50 NG/ML
MTD	METHADONE	300 NG/ML
mAMP	METHAMPHETAMINE	1000 NG/ML
MOR/OPI	MORPHINE/OPIATES	300/2000 NG/ML
OXY	OXYCODONE	100 NG/ML