

Single Panel Gabapentin Urine Drug Test Dip Card



Made in the U.S.A



99% Accuracy



Results in Minutes


Gabapentin abuse is a growing concern among treatment centers and recovery programs. This 'downer'-type drug is prescribed under the brand name Neurontin for the management of conditions like epilepsy, chronic pain, hot flashes, and sleep disorders. Gabapentin is an addictive substance which should be administered with care to prevent abuse. Our Single Panel Gabapentin Urine Drug Test Dip Card detects Gabapentin metabolites in urine within minutes to help detox specialists stay on top of trending drug abuse.

EACH KIT INCLUDES

- One Single-Panel GAB Dip Card
- Package Insert and Instructions

PRODUCT FEATURES:

- Detects Metabolites of Gabapentin, Pregabalin, and Vigabatrin
- Instant Results in 5 Minutes or Less
- 99% Result Accuracy
- Forensic Use Only
- 25 Dip Cards Per Case
- Made in the USA

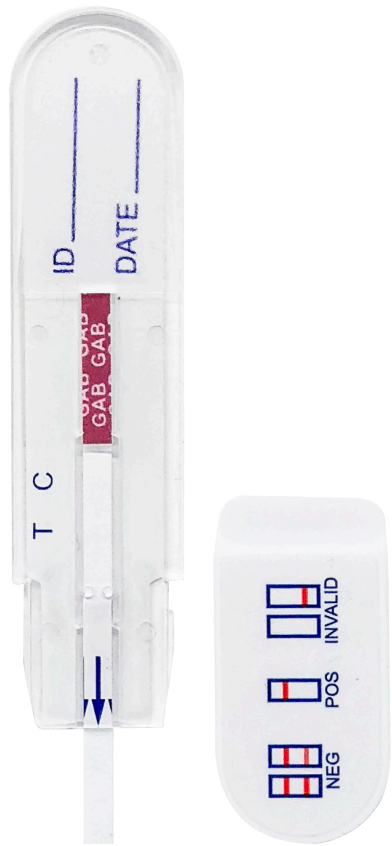


INSTRUCTIONS

1. Collect a urine sample with a clean, dry urine collection cup (not included). Bring refrigerated samples to room temperature (15°-30°C) before using test.
2. Remove test device from pouch. Label with Patient ID and use immediately.
3. Remove cap from the end of test card. With arrows pointing downward, immerse the strip portion of device into specimen for a minimum of 10-15 seconds. NOTE: For best results, DO NOT allow plastic panel to contact urine specimen.
4. Replace cap and place test card on a flat surface. Wait for colored line(s) to appear. Read result(s) after 5 minutes. DO NOT interpret test after 8 minutes or longer.

INTERPRETATION OF RESULTS

NEGATIVE:

A colored line next to the Control Region (C) and a colored line next to the Test Region (T) indicates a Negative result for Gabapentin, regardless of color intensity.

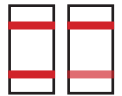

PRELIMINARY POSITIVE:

A colored line next to the Control Region (C) and no line next to the abbreviated Test Region (T) indicates a Preliminary Positive result. Confirm Positive test results with Liquid Chromatography/Mass Spectrometry (LC/MS) method.



INVALID:

If no line appears next to the Control Region (C) after ten minutes, the result is invalid or inconclusive. Discard the drug test and administer a new one.



IMPORTANT INFORMATION

Do not use after the expiration date.

The test device should remain in the sealed pouch until use.

Results are preliminary and positive results do not always mean that drugs are present in the donor. Certain foods and medicines, diet plan drugs, and nutritional supplements may yield a false positive test result with this product. Similarly, if the sample is diluted or the sample is adulterated, the product may yield negative results and not detect a particular drug. Laboratory tests confirm the preliminary results.

TEST	DESCRIPTION	CUT-OFF LEVEL
GAB	GABAPENTIN (NEURONTIN)	1000 NG/ML

FOR STATE-OF-THE-ART PREVENTION OF MEDICATION ABUSE, USE THE GAB DIP CARD TEST

The GAB Urine Dip Card is a useful accessory for clinical rehabilitative programs. If patients have a problem with medications abuse, this specific instant immunoassay can help them along the right path. Using the Gabapentin Urine Drug Test Dip Card is as seamless and trustworthy as your currently preferred drug test. The GAB Urine Dip Card is a useful accessory for clinical rehabilitative programs. If patients have a problem with medications abuse, this specific instant immunoassay can help them along the right path. Using the Gabapentin Urine Drug Test Dip Card is as seamless and trustworthy as your currently preferred drug test.